

Hearing loss can isolate someone we love and care for, even with hearing aids or other battery-powered devices. In some cases of severe hearing loss, especially with the elderly, issues with batteries, cleaning, adjustment and the high cost are difficult to resolve. Also, background noise can be a barrier to communication.

You will find **HearMeDear** helpful in certain situations where other devices have been tried without success. It will transfer the full spectrum of sound and in a natural, low-tech way that can open up conversation and maintain important relationships.

*An old idea . . . newly discovered!*

For more information go to  
[www.hearmedear.com](http://www.hearmedear.com)

## HearMe *dear* Conversation Aid



*Your handy, no-battery  
solution to a very common  
problem!*

## How To Use

**Listening:** Assemble HearMeDear and hold the “Flat Ear Piece” against the ear. Sound hole should be placed over ear canal.

**Tube Length:** The full 48” tube is ideal for across-table or chair-to-chair conversation. If you are a care-giver that moves from patient to patient, the tube may be shortened to arm’s length and carried around the neck like a stethoscope.

**Speaking:** Hold the “Speaking Bell” close to the lips and speak in normal, soft conversational volume. Amplification increases the closer the bell is held to the lips. **Caution:** Don’t speak loudly or force breath through the tube. This may damage the listener’s ear.

Practice will quickly determine how close the bell should be held to the lips.



**Cleaning:** Wash all parts with soap and water or wipe clean with wet wipes.

## Assembly

1. Slip one end of tubing through the hole in handle.
2. Insert Ear Piece into end of tube.
3. Slide handle back against Ear Piece for snug fit.
4. Insert Speaking Bell into the opposite end of the tube.



**Caution:** Loud sounds or air forced through this device can cause severe ear damage, so please speak in normal conversational volume.

**Return Policy:** If you are not fully satisfied with this device, please return it within 30 days of purchase. For more information go to

[www.hearmedear.com](http://www.hearmedear.com)