

Your hearingcare professional:

029-0149/0602 Printed in Switzerland © Phonak AG All rights reserved

PHONAK
hearing systems

PicoForte3

Operating Instructions

Gebrauchsanweisung

Mode d'emploi

Manuale d'istruzioni

Modo de empleo






Manual de instruções

Bruksanvisning

Betjeningsvejledning

Gebruiksaanwijzing

CE
0301

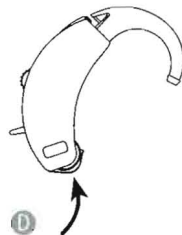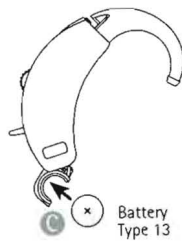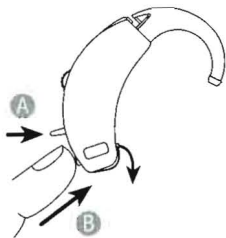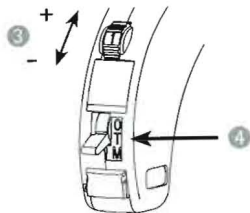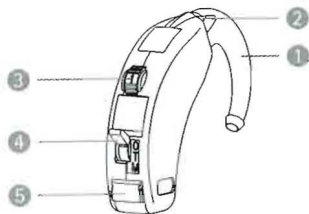

Congratulations on your choice of PicoForte3 hearing instrument(s). PicoForte3 is a quality Swiss-made product from Phonak, a name that stands for total reliability and technological innovation in the service of people. Used and looked after properly, your PicoForte3s will give you many years of trouble-free hearing and service.

The Phonak Team

Instrument description

1. Integrated ear hook
2. Microphone aperture
3. Volume control wheel
4. M-T-O switch
5. Battery compartment

Replacing the batteries

- A) Move the M-T-O switch to the "O" (Off) position.
- B) Open the battery compartment.
- C) Insert a type 13 battery so that the + sign on the battery (flat side) corresponds to the + mark in the battery compartment.
- D) Push the battery compartment gently back into position.

Important: Handle the battery compartment gently, do not use force. If there is any resistance, ensure that the battery is correctly inserted in the battery compartment. The door will not close properly if the battery is upside-down.

M-T-O switch (4)

Main switch with three positions:

- "O": The PicoForte3 is switched off.
- "T": The tele-coil is activated.
- "M": The microphone is switched on.

Volume control (3)

To increase volume, turn the control upward; to decrease volume, turn the control downward.


Switch position "T" (M-T-O switch)

"T" stands for tele-coil.

This function allows you to use a hearing instrument compatible telephone receiver, or to take advantage of a special loop conductor system which is frequently installed in schools, churches, theatres etc. For further information please consult your hearingcare professional.

Accessories

The audio shoe allows you to use various additional devices – for example Phonak's MicroLink wireless communication system or wireless television accessories – with your hearing instrument. Please consult your hearingcare professional.

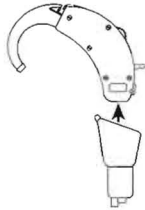

Child safety device

To protect children from the dangers of playing with batteries, your hearing instruments can be fit with child safety devices. Ask your hearingcare professional for details.

Important points

1. Always use new batteries as replacements. Your hearingcare professional will dispose of the old ones for you.
2. If you will not be using the hearing instruments for a long period of time, be sure to remove the batteries.
3. When instruments are not in use, store them safely in the case.

4. Protect them from excess moisture and heat. Take care when bathing and swimming. In summer, do not leave instruments near windows or car windshields.
5. Daily cleaning and a drying system are highly recommended. Your hearingcare professional will be glad to advise you.
6. Hair spray, creams and make-up can damage hearing instruments. Remove instruments before applying cosmetics.
7. If your PicoForte3s fail to operate after you have correctly inserted new batteries, contact your hearingcare professional for advice.

Warning: Never leave your instruments and/or batteries where small children or house pets can reach them. If batteries are swallowed, promptly seek medical attention.

Important: Use only hearing instruments that have been specially adjusted for you by a hearingcare professional. Other instruments may be ineffective or, in certain cases, even damage your hearing.

Service and Warranty

Phonak offers you a comprehensive global warranty which becomes effective on the date of purchase. Please ask your hearing healthcare professional about the details and duration. This warranty covers any repairs due to defects in material and/or workmanship. The warranty does not cover damage from improper handling or care, exposure to chemicals, immersion in water or undue stress. Damage caused by third parties or non-authorized service centers renders the Phonak warranty null and void. This warranty does not include any services performed by a hearing professional in their office. This warranty applies to the Phonak products listed below:

Serial number - right:

Serial number - left:

Date of purchase:
