



**ZENITH**

*The Zenith "75" Radionic Hearing Aid is a superb new instrument for the hard of hearing. It was developed and perfected in the laboratories of Zenith Radio Corporation, and is the result of engineering skills and experience gained in more than 30 years of specialization in Radionics Exclusively.*

**ZENITH RADIO CORPORATION**  
HEARING AID DIVISION

5801 Dickens Ave., Chicago 39, Illinois

**ZENITH RADIO CORPORATION** of New York  
527 West 34th Street • New York 1, N. Y.

**ZENITH RADIO CORPORATION** of California  
724 So. Spring St.  
Los Angeles 14, California



## HOW TO USE



THE

*Zenith* 75



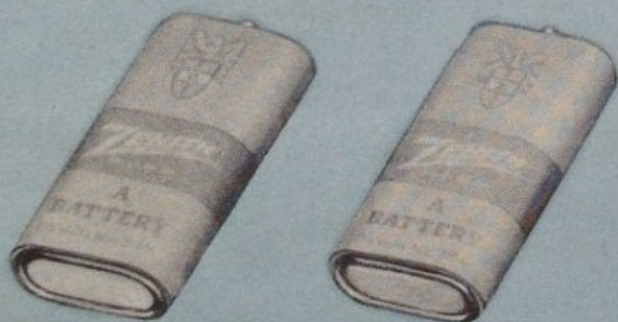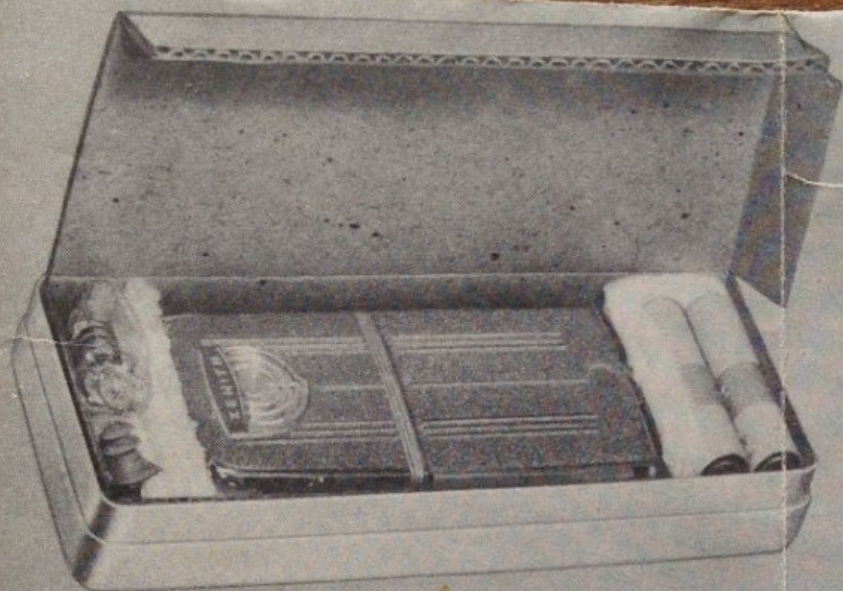

**1** The operation of your new Zenith "75" Radionic Hearing Aid is simple and easily understood. It is packed completely assembled with batteries. Assembled to the earphone is a medium size lucite tube and rubber tip; this size fits the majority of persons. Three additional lucite tubes and rubber tips of different sizes, as well as three vinylite washers and two spare "A" batteries, are included. A combination of one of these four lucite tubes and one of the four rubber tips will fit your individual requirements.

**NOTE:** Before using your Zenith "75" we urge you to read carefully the booklet "Learn To Hear Again."





**2** Before operating your hearing aid you will want to know about the different parts and the operating controls. Study the illustration carefully. Always wear the hearing aid with the microphone facing away from you and, if possible, on the side opposite from the ear in which the earphone is to be used.

**3** Moisten the tip slightly before inserting the rubber tip in the ear canal and make sure the Volume Wheel is in number 1 position. With a twisting motion the tip should enter the ear canal easily. A good fit is a snug fit. See enclosed folder for information on custom-molded plastic ear pieces.





BE SURE THIS  
WASHER IS IN PLACE



EARPHONE

VINYLITE  
WASHER

LUCITE  
TUBE

RUBBER  
TIP

**4** If the tip is too large or small for a proper fit, or if the earphone protrudes from the ear, a shorter lucite tube, and a larger or smaller rubber tip should be assembled to the earphone. The vinylite washer is a seal between the lucite tube and earphone. If the washer is not in place, you may hear a whistling sound when the hearing aid is turned on.




**5** The ON/OFF switch is part of the TONE CONTROL WHEEL. The instrument is OFF when the arrow points to OFF on the TONE CONTROL WHEEL. To turn it ON, turn the wheel to one of the four numbered positions. Choose the position that helps you to hear best. When the hearing aid is not in use, turn it off by setting tone wheel at OFF. This insures battery life.



**6** Adjust the VOLUME CONTROL WHEEL back and forth to the proper volume for your own needs. Best results will be obtained if the VOLUME CONTROL WHEEL is adjusted each time the TONE CONTROL WHEEL is reset. Other suggestions to help you get the most help from your new Zenith Radionic Hearing Aid are contained in the booklet, "Learn to Hear Again."



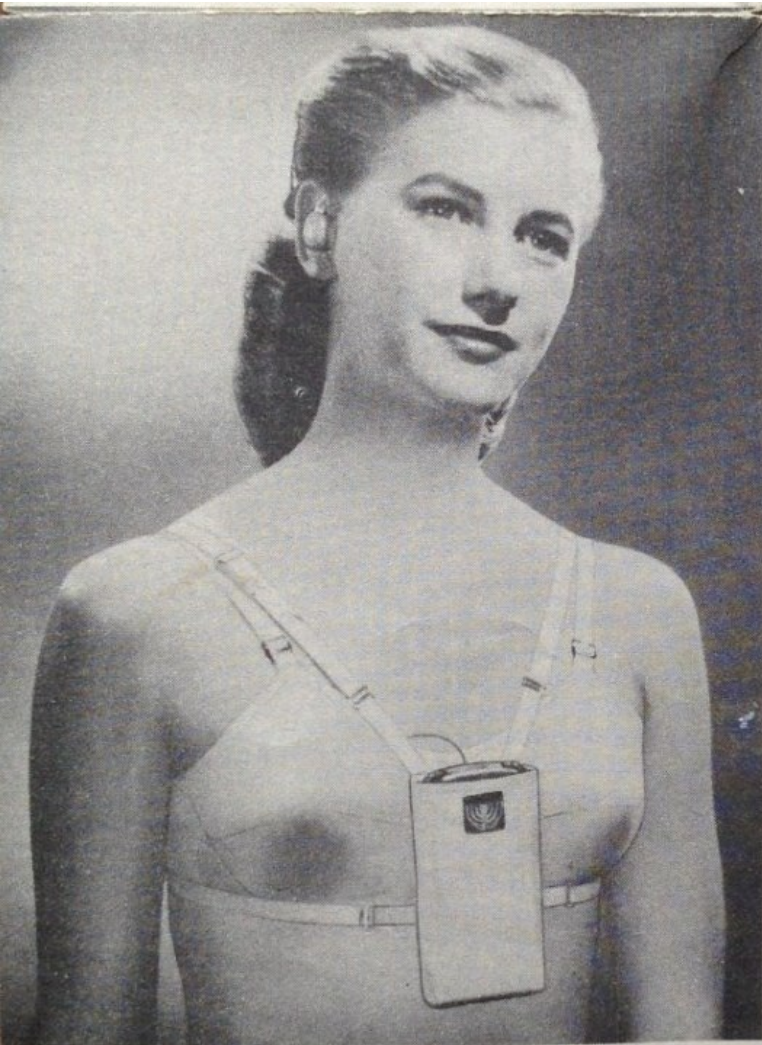


**7** When wearing your hearing aid in a garment, as shown, be sure the microphone faces away from the body. The earphone cord can be concealed under the garment straps. A choice of garments is available—see the accessory catalogue for full information.

**8** When wearing the hearing aid in your shirt or vest pocket, be sure to use the clips on each side of the case. Stretch the pocket tightly across the front (microphone) of the instrument. This will reduce clothing noise to a minimum. Under-clothing garments are also available for men.






**9** The proper way to remove the earphone cord is shown in the illustration. A damp (not *wet*) cloth will clean the cord. Do not use cleaning fluids . . . you may cause serious damage.

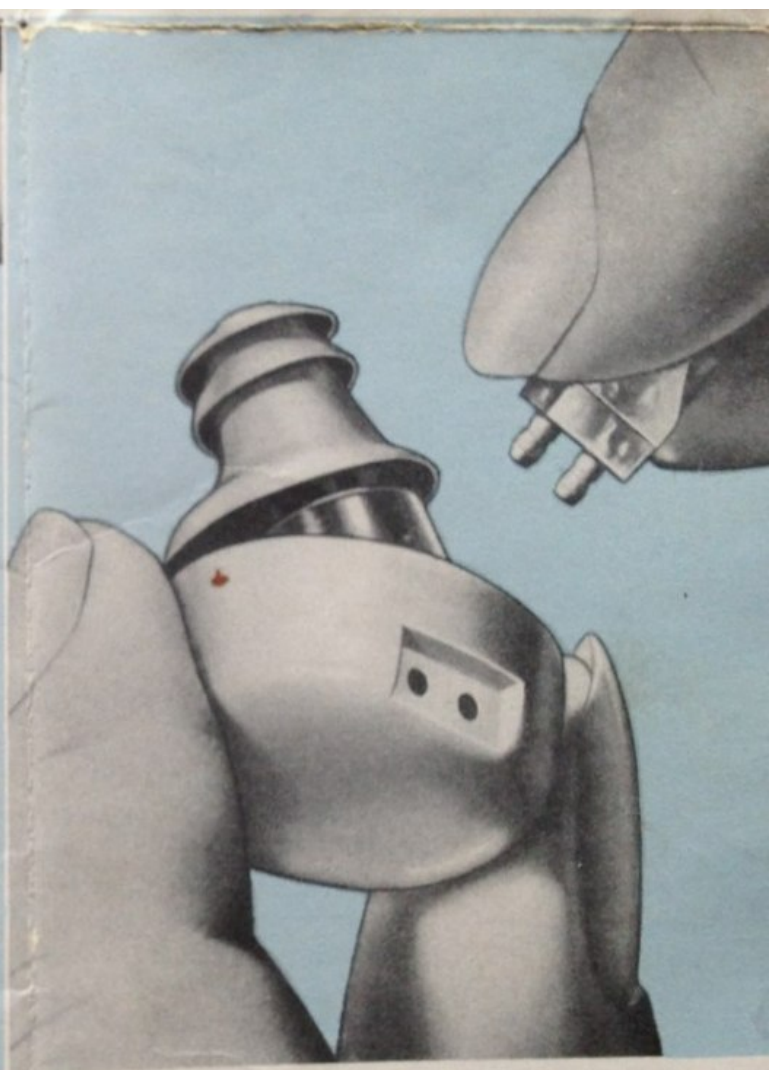

**10** Hold the earphone and cord tip as shown above. A slight pull will separate them. Never pull on the cord. Clean the instrument and earphone with a dry cloth. Be careful that you do not get moisture in the earphone.



**11** To change the batteries, first remove the lower portion of the back of the instrument, as shown. A slight outward pull on the metal sides will release the back.

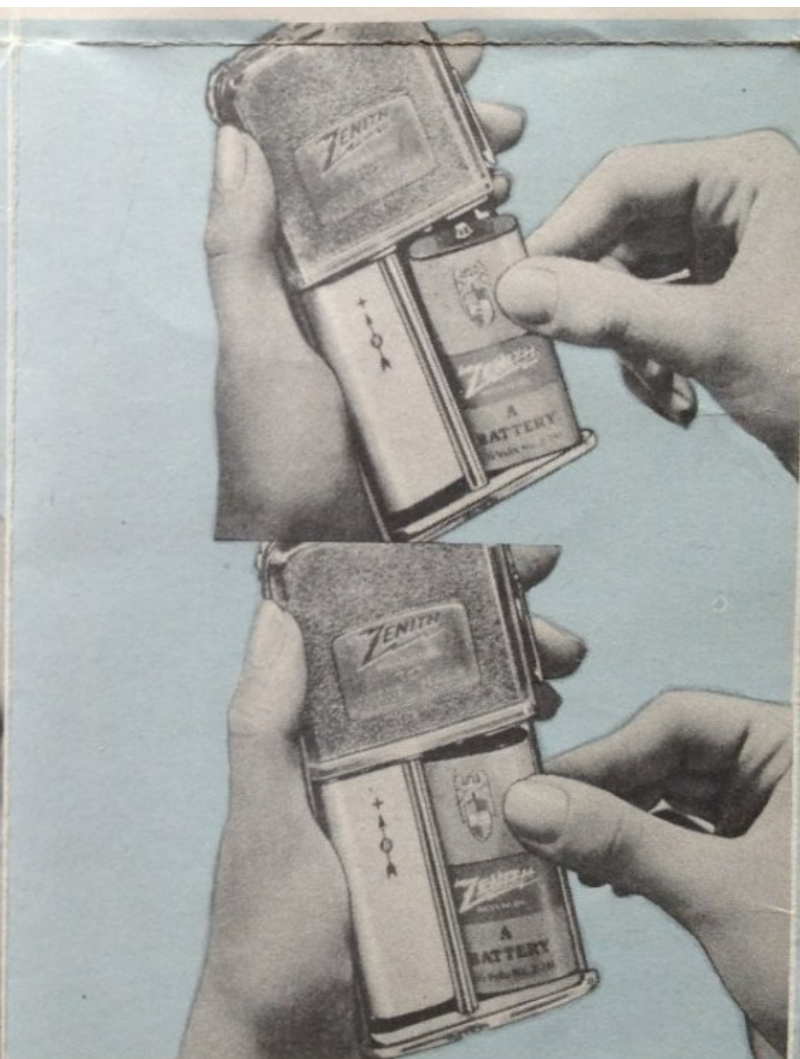

**12** To replace the "A" battery, lift it out of the instrument, as shown in the upper illustration. Install the new "A" battery with the printed matter right side up. This will insure proper installation of the battery. Ordinarily three or four "A" batteries will be used up before the "B" will have to be changed.





**13** A slight twisting motion will release the "B" battery. Lift the "B" battery straight out as shown above. Always replace the "B" battery with the arrow pointing toward the top of the instrument, press lightly on the side of the battery, then snap in place.



**14** To replace the back cover, insert the bottom edge first, and then snap the cover into place as illustrated. Before replacing the back, be sure the "A" and "B" batteries are installed with the printing right side up or the instrument will not operate.



## MAINTENANCE

15

The Zenith "75" Radionic Hearing Aid is ruggedly built, and it will stand up well under all normal use. However, it is a delicate precision instrument, and, for this reason, it cannot be treated carelessly. Naturally, it will not ordinarily withstand dropping or exposure to extreme moisture or high temperatures.

Handle your Zenith "75" Hearing Aid as carefully as you would a fine watch or eye glasses. Never allow it to lie in the hot sunlight, on a radiator, or in the immediate vicinity of any source of high temperature. Do not leave it in a closed automobile exposed to the sunlight.

*The guarantee does not cover such abuses.*


## SERVICE

16


If you will follow carefully the instructions included with your Zenith "75" Hearing Aid, and give it proper care, it will give excellent service. Like any precision-built instrument or mechanism, however, the time will come when it will need service to operate satisfactorily. If your Zenith "75" Hearing Aid some time fails to operate to your satisfaction, and if the suggestions given in the instructions do not correct the difficulty, send the instrument to us for service, in accordance with the Warranty and Five Year Service Policy as explained to the right.



## HOW TO RETURN THE ZENITH "75" HEARING AID FOR SERVICE

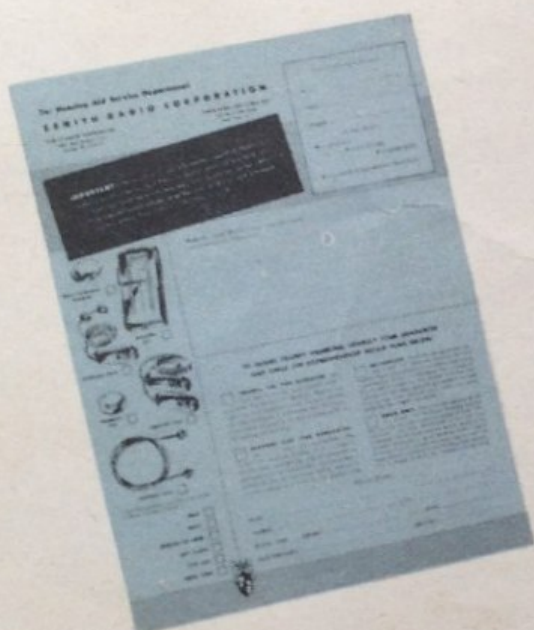

**17** The original carton in which you received your Zenith "75" Hearing Aid was carefully designed to insure safe transit, and it has received the approval of the post office authorities. Be sure to keep the aluminum case and the cardboard protective liners. Use them when returning your hearing aid for service. Enclosed with the hearing aid is a Service Form (see above), that will enable you to give us information needed to service your instrument properly and promptly. Be sure to keep this form.

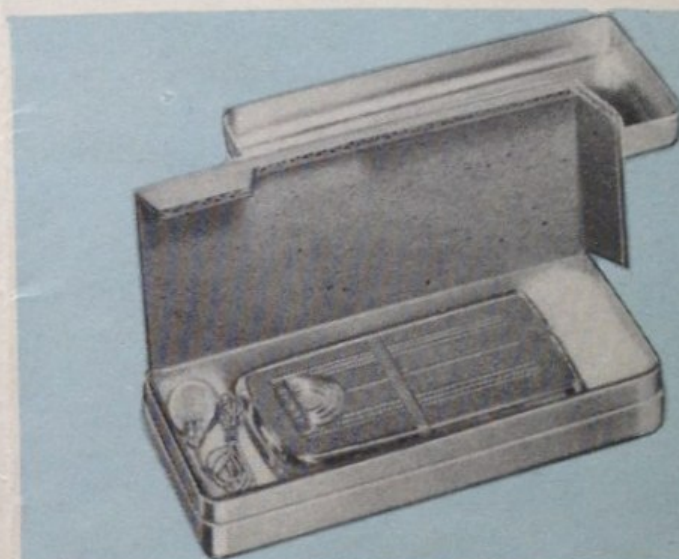

**18** When returning the instrument, it is to be packed in the shipping container, as illustrated above. The hearing aid case is placed in the center portion of the container, and the cardboard folded over carefully, as it was originally packed. The earphone and cord are placed in the section at one end of the box. Use tissue paper or similar packing, to completely fill battery space at the other end of the container. (Do not return batteries.) Place the metal lid down firmly.

Fill out the Service Form completely. If it is necessary to supplement this with a letter





of additional information, be sure your name and address appear on this letter as well as on the Service Form.

The service instructions are to be placed in the "envelope" section of the gummed return form which is included with your instrument.

**19** Moisten the gummed surface, and paste the entire mailing wrap on the top of the metal container. Tie cords firmly around the case in both directions.

**20** When mailing, you will notice that two positions are provided for postage. One is for first class postage covering your instructions which go in the special envelope section; the other is fourth class Parcel Post for the instrument itself. In this way, you save the expense of first class postage on the entire package. Be sure that your return address appears on the label, and that the package is addressed to Zenith Radio Corporation or its authorized sales or service offices or subsidiaries. Special gummed, addressed stickers are included for your convenience in addressing.



## Warranty

Each Zenith "75" Radionic Hearing Aid (with the exception of batteries, earphone cords and battery cords) is guaranteed against defects in workmanship or material for a period of one year from the date of purchase from Zenith Radio Corporation or its subsidiaries.

Our obligation under the above guarantee is limited to making good at our factory or offices of our subsidiaries, any defective part or parts of the hearing aid (with the exception of batteries, earphone cords and battery cords) which shall be returned to us, transportation charges prepaid, within the one year period and which shall be found to our satisfaction to have been thus defective. A charge of \$1.50 will be made to cover the cost of mailing and handling.

Earphone and battery cords are guaranteed against defects in workmanship or material for a period of 90 days from date of purchase.

Batteries are not guaranteed and shall not be returned. Neither does our guarantee cover damage to the hearing aid caused by corrosion or other chemical action of the battery fluids. (To avoid the possibility of this type of damage, do not leave exhausted batteries in the amplifier unit.)

## Five Year Service Policy

Any time after the expiration of one year and within five years from the date of purchase of a Zenith "75" Radionic Hearing Aid, Zenith Radio Corporation agrees, upon the return to it, or its subsidiaries, transportation charges prepaid, of any such hearing aid and the payment of the sum of \$12.50, to place the same in its original operating condition, including the replacement of all questionable parts of the complete hearing aid (exclusive of batteries, earphone cords and battery cords) and also to renew the guarantee of the hearing aid for the same period and subject to the same conditions as the original guarantee.

The foregoing warranties and service policy are specifically in lieu of all other agreements expressed or implied. We neither assume nor authorize any representative or other person to assume for us any other liability in connection with the sale of Zenith "75" Radionic Hearing Aids.

The warranties and service policy shall not apply to any Zenith "75" Radionic Hearing Aid which has been repaired or altered outside of our factory or offices of our subsidiaries, or which has been subject to misuse, negligence, or accident, or which has had the serial number or name defaced or removed.

The foregoing warranties and service policy extend only to the original consumer purchaser of the Zenith "75" Radionic Hearing Aid and are not assignable or transferable, and do not apply to persons living outside the continental United States of America, even if the instrument has been purchased in the United States.

ZENITH RADIO CORPORATION.